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**RESPIRADORES DESECHABLES
MODELO 4200, 4201,
4600 & 4601**

MOLDEX
Ideas that wear well.®

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Nueva y Exclusiva SmartStrap™

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Moldex-Metric, Inc.
CULVER CITY, CALIFORNIA, U.S.A.
+1 (800) 421-0668

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	PROTECTION ¹	RESPIRATOR	CAUTIONS AND LIMITATIONS ²	
		4200	4201	
844-5398	N95	X		A B C J M N O P
844-5397	N95		X	A B C J M N O P

1. Protection

N95 Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.
B- Not for use in atmospheres immediately dangerous to life or health.
C- Do not exceed maximum use concentrations established by regulatory standards.
J- Failure to properly use and maintain this product could result in injury or death.
M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configurations as specified by the manufacturer.
O- Refer to user's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
P- NIOSH does not evaluate respirators as surgical masks.

Moldex-Metric, Inc.
CULVER CITY, CALIFORNIA, U.S.A.
+1 (800) 421-0668

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	PROTECTION ¹	RESPIRATOR	CAUTIONS AND LIMITATIONS ²	
		4200	4201	
844-5636	N95	X		A B C J M N O P
844-5637	N95		X	A B C J M N O P

1. Protection

N95 Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.
B- Not for use in atmospheres immediately dangerous to life or health.
C- Do not exceed maximum use concentrations established by regulatory standards.
J- Failure to properly use and maintain this product could result in injury or death.
M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configurations as specified by the manufacturer.
O- Refer to user's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
P- NIOSH does not evaluate respirators as surgical masks.

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MOLDEX-METRIC, INC.

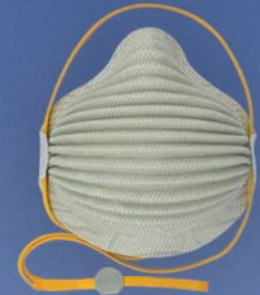
10111 W. Jefferson Blvd., Culver City, CA 90232
Tel: +1 (800) 421-0668 or +1 (310) 837-6500
Fax: +1 (310) 837-9563
Email: sales@moldex.com www.moldex.com

MEXICO / LATIN AMERICA

MOLDEX-METRIC, INC

Calz. Los Arcos 168 – 3
Col. Loma Dorada
Querétaro, Qro. 76060, México
TEL: +52 (442) 234-1230
FAX: +52 (442) 234-1231
E-mail: ventasamlat@moldex.com

MOLDEX
Ideas that wear well.®

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Printed on Recycled Paper

ISO 9001:2008
MOLDEX
REGISTERED

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Moldex, Ideas that wear well, Dura-Mesh, AirWave, FlexFit y el logo de PVC-Free son marcas registradas. Patente en trámite. El diseño de la malla es una patente de Moldex-Metric, Inc. ©2012 Moldex-Metric, Inc.