

2200GN95

SERIES HEALTHCARE PARTICULATE RESPIRATOR & SURGICAL MASK



EASY FIT, MORE COMFORT

The molded nosebridge with soft foam nose cushion provides the wearer an easy comfortable fit. There's no metal noseband to adjust, and the seal is maintained without pressure points.

LAST LONGER

The Dura-Mesh® shell, exclusive to Moldex®, protects the filter media from contaminated hand or glove contact. The shell resists collapsing, even in heat and humidity so the respirator holds its shape longer, increasing the life of the mask and comfort of the wearer.

NON-LATEX STRAPS


The non-latex straps don't irritate. They don't require prestretching, and they are securely attached.

APPLICATIONS

Surgery, hospitals and healthcare facilities, nurse/hospice care, correctional facilities, police, fire and rescue teams, EMS/EMT, homeless shelters, occupational health facilities, school healthcare offices.

FEATURES

- Both an N95 respirator and surgical mask (Latex Free).
- 100% PVC-Free.
- Meets CDC guidelines for protection against TB.
- NIOSH certified to have a filter efficiency of 95% or greater against non-oil based particulates.
- Dura-Mesh® shell resists collapsing in regular use and even under hot and humid conditions.
- Contour molded nosebridge for extra comfort.
- Non-latex straps.
- Soft foam nose cushion for an easy and comfortable fit.
- Softspun® inner lining is soft to the touch.
- Fluid resistant.
- Bacterial filter efficiency >99.9%.
- Available in 4 color coded sizes.

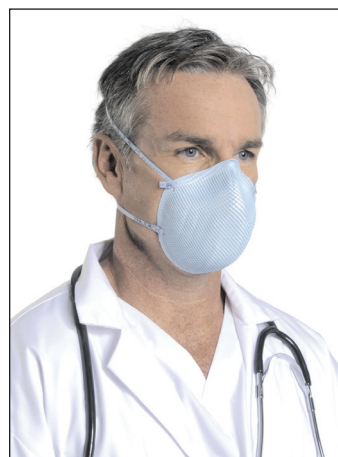

DESCRIPTION

1. The Moldex Healthcare Series N95 Particulate Respirator and Surgical Mask is designed to help provide respiratory protection for the wearer. This product has been tested¹ and certified by NIOSH as an N95 respirator and as having a filter efficiency level of 95% or greater against particulate aerosols free of oil. It is fluid resistant², disposable and may be worn in surgery or throughout the hospital.
2. This product contains no components made from natural rubber latex.

INTENDED USE

The various models of Moldex Type N95 Healthcare Particulate Respirators and Surgical Masks meet CDC Guidelines for TB Exposure Control within healthcare facilities. These devices are also intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids and particulate material.

CONTRAINDICATIONS

1. Not for use with beards, or other facial hair that prevents direct contact between the face and sealing surface of the respirator.
2. Eyewear must not prevent direct contact between the face and sealing surface of the respirator.
3. Not to be used on children.

¹ Tested in accordance with NIOSH 42 CFR 84

² Passed ASTM F 1862 @ 160mm Hg

RESTRICTIONS

1. Before use, a written respiratory protection program in accordance with 29 CFR 1910.134 must be implemented.
2. This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
3. Prior to each use, carefully inspect the entire respirator, including filter media and strap attachment area for tears and damage. Staple or strap attachment perforations do not effect NIOSH approval.
4. If respirator is damaged, distorted, a proper fit cannot be obtained or breathing becomes difficult, leave contaminated area as soon as possible and replace respirator.
5. If the respirator comes in contact with blood or fluids, including body fluids, leave contaminated area as soon as possible and discard and replace the respirator.
6. Do not alter, modify or abuse this respirator.
7. Store unused respirators in box/bag in a clean, dry, non-contaminated area.
8. Dispose of respirator according to your employer's policy and local regulations.
9. Do not reuse or store for reuse or hang around neck unless your employer specifically authorizes reuse.
10. When used for surgical procedures, discard after every use.
11. Use respirator before the "use by" expiration date printed on box/bag.

MOLDEX-METRIC, INC.


10111 W. Jefferson Blvd., Culver City, CA 90232
Tel: +1 (800) 421-0668 or +1 (310) 837-6500 Fax: +1 (310) 837-9563
Email: sales@moldex.com www.moldex.com

Moldex Technical Service Department: +1 (800) 421-0668 or +1 (310) 837-6500
Ext. 512/550

CANADIAN CUSTOMER SERVICE


TEL: +1 (310) 837-6500, Ext. 517
FAX: +1 (310) 837-9563

Moldex, Ideas that wear well, Dura-Mesh and Softspun are registered trademarks. The mesh design and the PVC-Free logo are trademarks of Moldex-Metric, Inc. U.S. Patents #D285,374 and #4,850,347. Made in U.S.A. 9703-714 REV C 10/09



Department of Health & Human Services
U.S. DEPARTMENT OF HEALTH & HUMAN SERVICES

Moldex-Metric, Inc.
Culver City, California 90232
+1 (800) 421-0668



THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC	PROTECTION	RESPIRATOR				CAUTIONS AND LIMITATIONS ²
		2070000-01	2070000-02	2070000-03	2070000-04	
844-4238	N95	X	X	X	X	ABCD MNOP

1. Protection

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.
B- Not for use in atmospheres immediately dangerous to life or health.
C- Do not exceed maximum use concentrations established by regulatory standards.
D- Failure to properly use and maintain this product could result in injury or death.
E- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
F- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
G- Refer to User's Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
H- NIOSH does not evaluate respirators for use as surgical masks.

PACKAGING

2210GN95-XS, Extra Small (Yellow Strap)
2211GN95-S, Small (White Strap)
2212GN95-M/L, Medium/Large (Blue Strap)
2217GN95-LP, Low Profile (Green Strap)
20 Respirators per Box
12 Boxes per Case

LIMITED WARRANTY IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. **Exclusive Remedies:** damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damages, or liability, direct, indirect, incidental, special or consequential, arising out of a sale, use or misuse, or the inability to use products by the user.



WARNING TO USER

1. Non-latex straps are used in the headbands of these respirators. However, individuals highly sensitive to natural rubber latex may have an allergic reaction.
2. **This product does not eliminate the risk of contracting any disease or infection.** Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.
3. This product has not been sold with warning or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact the Moldex Technical Services Department at +1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/550.

DISTRIBUTED BY:



MOLDEX
Ideas that wear well.